


Bluetooth Remote Control User's Manual – Flytemax2

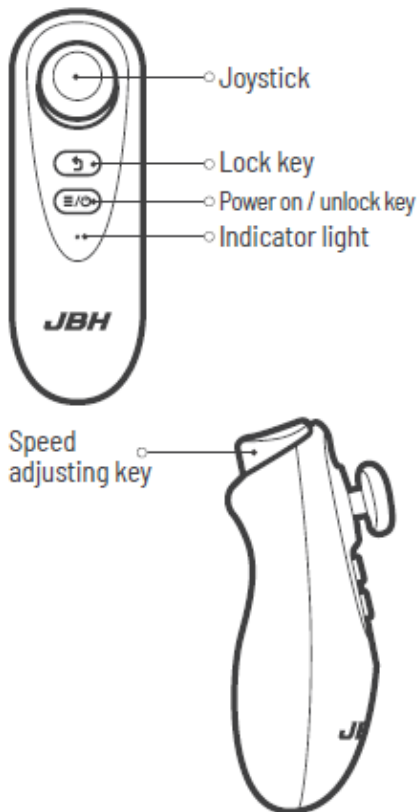


Function of keys on remote controller


Please keep the user's manual and read through the content before operation.

Step Of Operation

Have the powerchair and remote controller power on, then complete the connection.


To push the joystick to control the direction of the powerchair. Push the joystick to the front for forward and back for backward. Push the joystick to the left to turn left and to the right to turn right.



Powerchair Lock Key: Push down the  key to lock the powerchair.

Power On/Unlock Key: Push the  key to go into normal movable status.


Speed Adjusting Key: Push down speed adjusting key to find you suitable selection of speed. Cycle adjust speed among 3 sections.

Powerchair Awake: Push down the  key to wake up the powerchair.






Remote Control Automatic Shut Off: Automatic shutdown of remote controller is set to 20 seconds after the last operation. Touch  key again for restart, the automatic connection with the powerchair will be engaged.

Note: After pushing down  key, the powerchair is engaged to lock status, it is impossible to unlock the powerchair through power on or off. Simply push the  key to unlock powerchair of the remote controller. Please keep the remote control clean and dry.

First Time Use Of Remote Controller

Power On: Load 2 battery (No.7) on the powerchair. Push the  key into normal movable status.

Bluetooth Connection of Powerchair and Remote Controller

1. Power on powerchair.
2. Long push the horn of the powerchair for 10 seconds, or until the Bluetooth icon is flashing on the powerchair display, which allows the powerchair to enter matching mode.
3. Push the  key and the  key, the indication light will start flashing on the remote controller, this means that it is entering into Bluetooth connection. The Bluetooth connection is completed when the light stops flashing. Push the  key on the remote controller when the light indicator is on to connect. If the light indication is not on, this means it has failed to connect. Repeat the steps above if the connection fails.
4. Once connection is completed, simply push  key on the remote controller, if the light indicator is on this means the connection is completed. If the light indicator is not on, this means the connection has failed. Touch  again for restart until the light indicator comes on.

Statement

Remote controller cannot 100% accurately replace the manual controller. It is suitable for the caregiver to use in a safe environment. Never have other people operate with the remote control.