

Dealer Bulletin # DB2024-08

Dual Battery Fitting Mount

This bulletin explains how to install the dual battery mount on the Flytemax2 Powerchair.

This is the battery mounting slot of the standard version of Flytemax2, which only has one. The accessory battery mounting slot also has two, which can be used to install two batteries.

Instructions for use are as follows:

1. Remove the battery mounting slot on Flytemax2.
2. Install the dual battery mounting wipe in the original fixed position.
3. Connect the power supply.



SINGLE BATTERY MOUNT



DUAL BATTERY MOUNT