


Dealer Bulletin # DB2021-13

Pill Dispenser Medical Timer (Bluetooth)


APP Installation Instructions

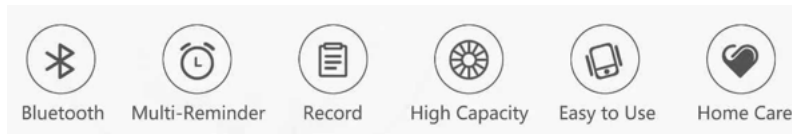

To set up the Bluetooth for the Bluetooth Pill Dispenser, you will need to do the following steps:



1. Turn on Bluetooth & GPS
2. A scan interface will show up, click the “+” symbol.
3. Scan the QR Code at the back of the Bluetooth Pill Dispenser to download the APP.

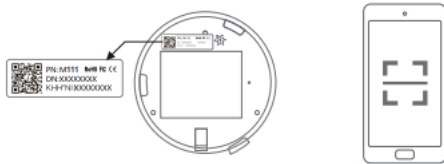

4. An audio notification will be prompted from the device once connected.
5. Time on apps will be synchronised to the device.