

Dealer Service Bulletin # DSB159 Seat Removal (Outlaw)


Follow the below steps to safely remove the seat from the Outlaw Mobility Scooter

With the seat in the forward facing, straight position, pull the lever on the Left hand side upwards and spin the seat one “click” to the right (45 degrees)



Once the seat is in the 45 degrees position the seat is able to be removed by lifting it straight up

Note : The seat has a safety latch underneath that prevents the seat from being removed in a forward facing position. By turning the seat to 45 degrees it releases the latch overlay and allows the seat to be removed.

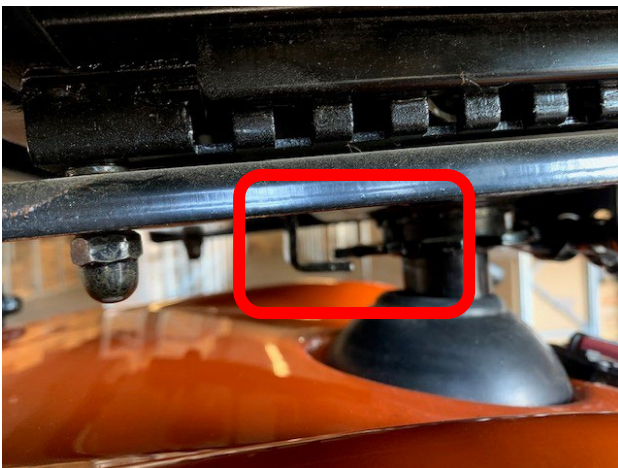

To put seat back onto the scooter, ensure the seat base is at a 45-degree angle when placing seat on post. Once in that position, wiggle the seat slightly to ensure the seat has bottomed out and that you are able to lift the lever on the left-hand side. Once the lever is lifted you can then swivel the seat back to the center position and release the lever ensuring it locks into place.