



USER MANUAL



Bottom Wiper (EG0011)

Copyright Zoom Mobility

User Manual Booklet 2025

Pictures used are for illustration purposes only

Please read the instructions carefully before using this product.

The Zoom Mobility Bottom Wiper is ideal for people with limited hands movement, dexterity, individuals recovering from surgery & arthritis. This product is specially designed to reduce straining and bending when using the toilet. Do not use this product without reading and understanding these instructions. If necessary, contact a healthcare professional or technical personnel.

Preface

- I. Check each part carefully before use. If anything becomes broken, do not use.
- II. Keep these instructions in a safe place for future reference.

Instructions

1. Pull around 24” of toilet paper (around 6 sheets) and fold them in half.
2. Hold the bottom wiper and tuck one end of the toilet paper into the oval access.
3. Wrap the toilet paper all around the top part of the wiper and then tuck the other end of the toilet paper into the same recess.
4. The toilet paper will then be held in place for cleaning.
5. It is recommended to be used between the legs from front to back while sitting on the toilet.
6. Once finished, push the button at the end of the bottom wiper.

Care

Do not immerse the bottom wiper in water.

Clean with a damp cloth and mild soap.

Dry with a soft cloth.

For maximum hygiene, clean after each use.

FOR EXTERNAL USE ONLY

DO NOT USE IN THE SHOWER OR BATH

FOR HYGIENE PURPOSES, DO NOT SHARE BETWEEN PEOPLE

NOT TO BE USED ON ANIMALS OR YOUNG CHILDREN

Warranty

One-Year Warranty

- Bottom Wiper

Your Zoom Mobility product is fully guaranteed for one (1) year from the date of purchase against faults arising due to defects in the manufacture or materials. This warranty does not detract from it but is in addition to your legal rights.

Any attempt to open or dismantle this item renders the guarantee void on that item.

Not Covered Under Warranty

The guarantee does not extend to those items which may need replacement due to normal wear and tear, or damage to the product caused by misuse or accident for which Zoom Mobility or its agent cannot be held responsible. This warranty does not include service calls.