



USER MANUAL



Copyright Zoom Mobility

User Manual Booklet 2025

Pictures used are for illustration purposes only

**Dawn Clock
(EG0044)**

Please read the instructions carefully before using this product.


The Zoom Mobility Dawn Clock has been designed to support independence and wellbeing, particularly for people living with disabilities or health conditions. For people who may have difficulty keeping track of time, the Dawn Clock can also act as a clear visual cue, helping to structure daily routines. Its large display, simple controls, and multiple settings make it user-friendly for individuals with limited mobility, vision, or memory challenges.

Preface

- I. Open the package and take out the product and all components.
- II. Check each part carefully before use. If anything becomes broken, do not use.
- III. Keep these instructions in a safe place for future reference.

Main Features

1. Shows date & time without abbreviations, easy to read.
2. 5 alarm settings, 3 medications reminder alarms, alarm volume adjustment.
3. Auto and manual brightness adjustment.
4. 10 language options.
5. Three user interfaces: digital clock UI, analogue clock + colour bar digital clock + digital photo frame features.

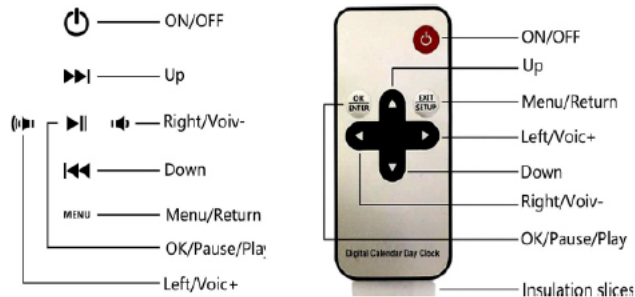

Display View

Choose display view by pressing middle button (OK) for 2 seconds to switch between view 1, 2 and 3.



Line 7-Volume Set

Press **OK** button or press **LEFT** and **RIGHT** button to choose the best volume level. Volume level from 0 to 30, 30 is the highest volume and 0 is mute.

Line 8-Alarm Set

Press **OK** button to enter the alarm clock setting.

There are 5 groups of alarms, set the time and alarm frequency. After you have finished the setting, press **MENU** to exit.

Line 9-Medication Reminder

Press **OK** button to enter the medication reminder setting.

There are 3 groups of medicine reminders (morning/afternoon/evening medicine reminder), set the time and reminder frequency. After you finish the settings, press **MENU** to exit.

Line 10-Factory Reset

Press **OK** button, it will bring up **YES** or **NO** options for a factory reset.

Choose **YES**, all settings come back to default factory mode.

Choose **NO**, to keep all your current settings.

Press **MENU** to exit.

Digital Photo Frame Features

- a. Put images or videos into the U disk and insert the U disk on the clock USB port (when you are in the clock interface), it will play photo and video automatically.

Image resolution should be 1024*768/1024*600.

Support image format: JPG, JPEG.

Tip: Please use a genuine USB flash drive, maximum support capacity is 32GB.

Settings

Press **MENU** button to enter screen menu. Use **UP** and **DOWN** button to highlight the line to apply changes. Use **LEFT** or **RIGHT** button to underline the text for adjustment. When underlined, use UP and DOWN button to make changes. Press **MENU** button to close changing mode.

Language Set	<	English	>
Time Set	<	12:53:30	>
Date Set	<	31-7-2020	>
Time Mode	<	24 Hour	>
Date Mode	<	Day-Month-Year	>
Auto Dimming	<	Auto	>
Volume Set	<	30	>
Alarm Set			>
Medicine Reminder			>
Factory reset	<	V3.2.3-20200408	>

Menu Settings

Line 1-Language Set

Press OK button and choose preferred language using LEFT or RIGHT button. Possible options English / French / German / Dutch / Italian / Polish / Spanish / Welsh.
Press MENU button to exit from menu screen.

Line 2-Time Set

Press OK button to enter the time set.
Press LEFT or RIGHT button to underline the hour <07:10> change the hour by using UP or DOWN.
Press LEFT or RIGHT button to underline the minutes <07:10> change minutes by using UP or DOWN button.
Press middle button OK to confirm changes and MENU button for exit menu screen.

Line 3-Date Set

Press OK button to enter the date set.
Before setting date select preferred Date Mode (see description under Date Mode).
After Date Mode is set press LEFT or RIGHT button to highlight and underline date/month/year.
Using UP or DOWN button to correct date/month/year.
Press MENU button to exit from screen menu.

Line 4-Time Mode <12 Hour / 24 Hour>

Press OK button or press LEFT and RIGHT button to choose 24hours/12hours option.
Press MENU button to exit from menu screen.

Line 5- Date Mode <Date-Month-Year, Month-Date-Year>

Press OK button or press LEFT and RIGHT button to choose one of two options <Day-Month-Year> <Month-Day-Year>.
Press MENU button to exit from menu screen.

Line 6-Auto Dimming <Auto/Mid/Low/Off>

Press OK button or press LEFT and RIGHT button to choose brightness level (Off/Mid/Low/Auto), press MENU button to exit from menu screen.

Warranty

One-Year Warranty

- Dawn Clock

Your Zoom Mobility product is fully guaranteed for one (1) year from the date of purchase against faults arising due to defects in the manufacture or materials. This warranty does not detract from it but is in addition to your legal rights.
Any attempt to open or dismantle this item renders the guarantee void on that item.

Not Covered Under Warranty

The guarantee does not extend to those items which may need replacement due to normal wear and tear, or damage to the product caused by misuse or accident for which Zoom Mobility or its agent cannot be held responsible. This warranty does not include service calls.