



Technical Service, please scan the following QR code.

USER MANUAL



Copyright Zoom Mobility

User Manual Booklet 2023

Pictures used are for illustration purposes only

Pill Dispenser Medical Timer (Automatic) (EG0024)

Please read the instructions carefully before using this product.

Whether you're using the medical timer pill dispenser to keep track of your own medications, or to assist or inform a family member, this Pill Dispenser promotes better health by alerting users and dispensing their medications at right times of the day and week. Organizing medication correctly, assigning dose times and determining what medications go into each slot is the responsibility of you, your care giver, and/or your healthcare provider

Preface

- I. Open the package and take out the product and all components.
- II. Check each part carefully before use. If anything becomes broken, do not use.
- III. Keep these instructions in a safe place for future reference.

Tips and Precautions.


- A low battery symbol will appear in the upper right corner of the LCD display when battery power is low. Be sure to change the batteries promptly to ensure the proper operation of the pill dispenser.
- Do not place the appliance near open flames or heat sources. Fire, electric shock, product damage, or injury might occur.
- Do not let children use or play with this device.
- Make sure to keep the pill dispenser on a flat, stable surface. Do not flip it upside down unless you are removing pills from their compartment. Please note, the internal pill container is removable for easy filling.
- Be careful not to drop the pill dispenser, as this could damage the product or the pills inside.
- Water can damage the dispenser, cause issues with the batteries, or otherwise adversely affect the medication contained inside. Do not immerse this pill dispenser in water or other liquids or expose it to liquids that might get splashed on it. Do not place it in an environment where there is excessive moisture or humidity.
- Do not fill the internal pill container unless it is completely dry.
- Never force the locking cover open.
- Never use mismatched batteries, or batteries with wrappings that have been torn or damaged in any way.

Product Features

1. Automatic Pill Alerts: The system will repeatedly flash the light indicator ring, rotate the internal pill container, and sound a beeping alarm every seven (7) seconds, until the unit has been fully inverted, for a period of up to 30 minutes, according to the schedule you set.
2. 28 Spacious Pill Compartments: The internal pill container features ample space in each compartment to accommodate multiple pills or tablets, so you can fit all the pills you need to take at a designated time in the same slot.
3. Clear, Removable Tray: Lift the internal pill container out of the main body of the device for easy refilling. The container automatically rotates according to the schedule you set.
4. Locking Cover: The powerful latch and the manual locking key help keep your medications secure and prevent messes or spills, unwanted tampering, accidental removal, or theft.


Parts and Controls

1. Top Cover	2. Cover Handle
3. Dispense Door	4. Dosage Viewer
5. Light Indicator	6. Dosage Ring
7. LCD Display	8. Button A.
9. Button B.	10. Button C.
11. Removable Tray	12. Cover lock
13. Battery Door	14. Battery Door Latch
15. Speaker	16. Coverlock

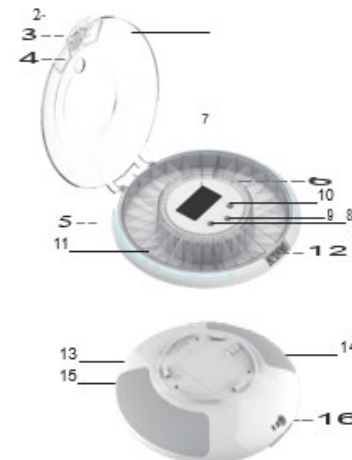

INSERTING BATTERIES

1. Turn the dispenser upside-down.
2. Slide Battery Door Latch and remove Battery Door.
3. Insert 4 x AA batteries in correct polarity and replace door. LCD Display will turn on.

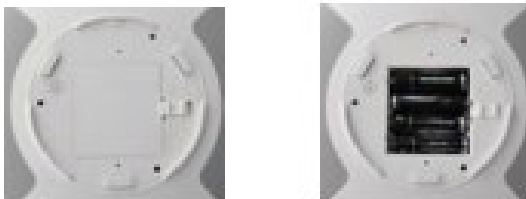


OPEN/LOCK TOP COVER

1. If locked, locate Cover Lock on bottom of unit and insert the included key, then turn clockwise to unlock.
2. Put Pill Dispenser on table, press Cover Handle to release the latch, then lift to open Top Cover.
3. To lock Top Cover, please close Top Cover first, press Cover Handle to get latched, then turn counterclockwise to lock with inserting the key from bottom side.



INSTALLING THE DOSAGE RING

1. Automatic pill dispenser comes with 6 different rings, corresponding to the number of times per day that medication needs to be dispensed. The rings are designed to help you in laying out medication in the internal pill container, so it is properly dispensed.
2. Set correct Dosage Ring on centre of tray and fold the tab down to secure in place.
3. Place pills in correct dosages in each

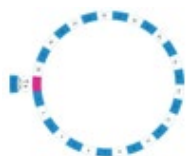

compartment.

* Do not place pills in compartments corresponding to red markings on Dosage Ring.

MEDICATION AND STOP ALARM

Both Pill Dispenser and smart phone would alarm when it's time to take medication.

Slide to open the Dispense Door (3) and tilt up Pill Dispenser to pour medications into your palm or a suitable container, the alarm and Light indicator (5) will then shut automatically.



LOW BATTERY AND BATTERY SAVING

A low battery symbol c: in screen indicates that the batteries are low and should be replaced to ensure proper function.

To save power, Pill Dispenser will stand by, and display will shut down if no operation in several minutes. To wake it up, just touch or slightly move Pill Dispenser.

SETTINGS WITH MECHANICAL BUTTONS

Open the Top Cover first as stated in "Open or Lock Top Cover" hereinbefore.

1. Setting the Time

- Press and hold button A for approximately 3 seconds until ":" symbol stops flashing.
- Press button B to adjust hour digits. Press button C to adjust minute digits.
- Confirm clock settings by pressing button A.

2. Setting Medication Times

- Press button A once, an "alarm clock" symbol and number "1" will be displayed.
- Press Button B to adjust hour digits. Press Button C to adjust minute digits.

Press and holding button B and C will cause the digits to speed up.

- To Cancel a previously set alarm, continue pressing button B until "-----" is displayed. This occurs after 23:00 or 11:00pm. Or pressing and holding button A until "-----" is displayed.



- After setting a desired alarm time , confirm by pressing button A. The next alarm number “2” will then be displayed.
- Repeat above steps 2 through to 4 to set alarm 2,3,4,5 and 6.
- After confirming alarm 6, display will return to clock mode.

SETTING THE TIME FORMAT

- Under clock mode, press Button C once to enter time format setting, then press Button C to switch between 12-Hour and 24-Hour time format.
- Confirm selected time format by pressing button A or just wait for approximately 15 seconds.

SETTING VOLUME CONTROL

- Under clock mode, press button C once to enter volume control setting, then press Button B to switch between 0, 1 and 2. "0" means mute, "1" means low volume, "2" means high volume.
- Confirm selected volume control by pressing Button A or just wait for approximately 15 seconds.

FCC COMPLIANCE STATEMENT

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

FCC Warning:
§ 15.21 Information to user.

Any Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.
§ 15.105 Information to the user.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that

interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.

-Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.

-Consult the dealer or an experienced radio/TV technician for help.

* RF warning for Portable device:

The device has been evaluated to meet general RF exposure requirement.

The device can be used in portable exposure condition without restriction.

CAUTION

Risk of explosion if battery is replaced by an incorrect type. Dispose of used batteries according to the instructions.

CORRECT DISPOSAL OF PRODUCT

This marking indicates that this product should not be disposed with other household wastes. To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle it responsibly to promote the sustainable reuse of material resources. To return your used device, please use the return and collection systems or contact the retailer where the product was purchased. They can take this product for environmentally safe recycling.

RADIO FREQUENCY

Frequency bands: 2402MHz to 2480MHzMax. Transmission power: 8dBm

Warranty

One-Year Warranty

- Pill Dispenser Medical Timer (Automatic)

Your Zoom Mobility product is fully guaranteed for one (1) year from the date of purchase against faults arising due to defects in the manufacture or materials. This warranty does not detract from it but is in addition to your legal rights.

Any attempt to open or dismantle this item renders the guarantee void on that item.

Not Covered Under Warranty

The guarantee does not extend to those items which may need replacement due to normal wear and tear, or damage to the product caused by misuse or accident for which Zoom Mobility or its agent cannot be held responsible. This warranty does not include service calls.

