



# USER MANUAL

Technical Service, please scan the following QR code.



Copyright Zoom Mobility  
User Manual Booklet 2023

## Shower Chair Adjustable Height

Please read the instructions carefully and use this product correctly.

## 1. Read Instructions

- I. Open the package and take out the product and all components.
- II. Check each part carefully before use. If anything becomes broken, do not use.
- III. Keep these instructions in a safe place for future reference.



## 2. Assembly

- I. Place seat face down on a table or counter with the seat lip near the edge of the table. Please note the triangle position. The triangle should be facing up.
- II. Align the leg frame that has the concave area on the underside with the screw holes on the seat. (As below).



- III. Align the next leg frame that has the concave area on the top side with the last two screw holes on the seat.

Hold both legs in the middle with one hand and align the centre hole where the concave sections are. With the other hand, fasten one **SHORT** hand knob screw making sure the washer goes on first so it's resting in the screw hold pocket (as above).

- IV. Align one L-shape back seat frame to the two screw holes on the seat. Fasten together using 2 washers and 2 **LONG** hand knob screws.



- V. Repeat Step 4 for the other L-shape back seat frame.

- VI. Attach the back support to both L-shape frames and fasten with 4 washers and 4 **SHORT** knob screws using only your fingers to tighten firmly as shown below.



- VII. Attach each armrest to each side where you see the round metal sleeves on the frame legs. The armrests should be at an angle pointing away from the chair. Make sure you hear an audible snap when securing the armrests.



- VIII. Attach the shower hanger on the one side of the rounded metal sleeves.



- IX. Check for stability. Check periodically to make sure all fixings are tight and secure, and legs are the same height.

