



USER MANUAL



Copyright Zoom Mobility

User Manual Booklet 2025

Pictures used are used for illustration purposes only

Toilet LED Night Light (EG0017)

Please read the instructions carefully before using this product.

1. Preface

- I. Open the package and take out the product and all components.
- II. Check each part carefully before use. If anything becomes broken, do not use.
- III. Keep these instructions in a safe place for future reference.

2. Instructions

Adjust the length

Adjust the length of the arm to the right size.



Install to the toilet.

Insert batteries and hang the light bowl onto the rim of the toilet.

Turn out the lights.

The light only works in the dark. When movement is detected, it will turn on and stay on for 2 minutes after the last movement. If the lights are on the light bowl will stay off saving the batteries for when you need them the most.



Set a colour or let them rotate

Set the button to the out position to cycle colours. Press the button in to set to a single colour when desired colour is visible.



Note: Your light bowl will work best if the sensor is facing the door. Some locations may be better than others.

3. How to Clean

Wipe with a damp cloth or cleaner. The sensor light is designed for daily indoor bathroom use only (Do not immerse in water or place directly under running water).

Warranty

One-Year Warranty

- Toilet Night Light

Your Zoom Mobility product is fully guaranteed for one (1) year from the date of purchase against faults arising due to defects in manufacture or materials. This warranty does not detract from but is in addition to your legal rights.

Not Covered Under Warranty

The guarantee does not extend to those items which may need replacement due to normal wear and tear or damage to the product caused by misuse or accident for which Zoom Mobility or its agent cannot be held responsible. This warranty does not include service calls.