



Technical Service, please scan the following QR code.



Copyright Zoom Mobility

User Manual Booklet 2023

USER MANUAL



Knee Walker

Please read the instructions carefully and use this product correctly.

1. Dimensions

Carbon Steel
Net Weight: 9.2kg




2. Read Instructions

- Open the package and take out the product and all components.
- Check each part carefully before use. If anything becomes broken, do not use.
- Keep these instructions in a safe place for future reference.

3. Assembly

a. Folding Buckle Installation

- Ensure that the trigger button is facing down and force the steering bay and the grip bar to close, so that the fixed pin correctly engages with the hole; and
- Rotate the trigger clasp upward to a vertical position and press down the handle of the trigger clasp to lock.

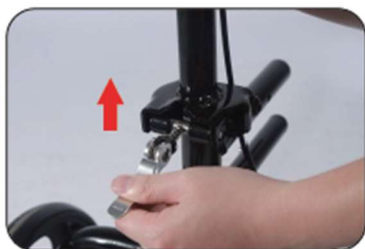

4. Combine Rear Wheel Frame and Main Frame

- Expand the rear wheel frame, pass through the knob screw, and lock it with the nut.
- Butt the rear wheel and the frame tube port together, press the marble button to insert the rear wheel frame into the frame tube, and adjust the appropriate gear (3 gear adjustment options).

- Install the hex screw and screw on the knob nut. Note: The hex screw head corresponds with the hex hole on the frame.



Screw in the screws and nuts



The marble pops out of the hole



5. Adjust The Grip Height (see below)

- Adjust the grip tube to the appropriate height, set the holes of the two tubes, install the hex screws, and screw on the knob nut. Note: the hex screw head corresponds with the hex hole on the frame.




6. Adjust The Knee Pad Height (see above)

- Insert the knee pad tube into the main frame.
- When you find a comfortable height setting, insert the star knob, and turn it clockwise until it is fully pinned in place. Note: Before using a knee slider, be sure to double check that the knee pad is fully secured and does not have any looseness.

7. Install Foot Pedals

- Choose the direction of the foot pedal, put it into the front foot tube, and put on the tube sleeve. The foot pedals can be removed and interchanged.

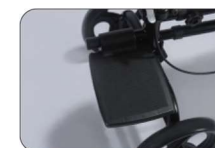

8. Hand Brake Operation

- To slow down or temporarily stop, use four fingers to slowly pull and engage the brake, until you slow down or temporarily stop.

