

Top Gun Mobility & Recliner Lift Chairs



USER MANUAL

FOR YOUR RECORDS

Please fill in your Top Gun Mobility Recliner Lift Chair information below. This information will be useful in the event

that you ever need to contact Top Gun Mobility concerning your recliner lift chairs.

Model _____ Serial

Number _____

Date of Purchase _____ Invoice Number _____

Provider Name _____

Company _____

Address _____

City _____ State _____ Postcode _____

Thank you for purchasing your Top Gun Mobility Recliner Lift Chair. We are honored you have chosen Top Gun Mobility for your mobility needs.

Intended Use:

The intended use of this device is to provide mobility to persons limited to a seated position that are capable of operating a Recliner Lift Chair.

Standard Packing List:

1 x Remote

1 x Chair



DISCLAIMER

We ask that you read this manual completely before operating your new Top Gun Mobility Recliner Chair. Top Gun Mobility is not and cannot be held responsible for any damage or injury incurred due to improper or unsafe use of the Recliner Lift Chair. Top Gun Mobility specifically disclaims responsibility for any bodily injury or property damage that may occur during any use that does not comply with applicable federal, state, or local laws or ordinances.

Table of Contents

INTRODUCTION	1
SPECIFICATIONS.....	2
GENERAL GUIDELINES.....	3
MODIFICATIONS	3
PINCH/CRUSH HAZARDS.....	4
DEGREE OF PROTECTION/MODE OF OPERATION	4
ELECTROMAGNETIC AND RADIO FREQUENCY INTERFERENCE (EMI/RFI)	4
MOTOR VEHICLE TRANSPORT	5
IMPORTANT SAFETY INSTRUCTIONS	6
SERVICING OF DOUBLE INSULATED PRODUCTS	7
ASSEMBLY AND DISASSEMBLY	8
CORD AND TRANSFORMER CONNECTIONS.....	8
CHAIR ASSEMBLY	9
OPERATING YOUR POWER LIFT CHAIR	10

OPERATING YOUR 5 MOTOR POWER LIFT CHAIR	11
ENVIRONMENTAL PROTECTION	12
IMPORTANT SAFETY INSTRUCTIONS	12
WARRANTY	13
<i>NOT COVERED UNDER WARRANTY</i>	1
<i>COMMERCIAL USE</i>	1

INTRODUCTION

CONGRATULATIONS on the purchase of your new Top Gun Mobility Recliner Lift Chair. We are excited to provide you one of our Innovative Recliner Lift Chairs. Our Recliner Lift chairs are designed to provide Convenience, Comfort and the Assuredly High-Performance level combined with a long-life cycle.

This User Manual contains important information concerning the proper maintenance and safe operation of your Recliner Lift Chair. Please read this manual thoroughly and carefully prior to operating your new Recliner Lift Chair to familiarise yourself with all operation instructions.

By accepting delivery of this product, you promise that you will not change, alter or modify this product or remove or render inoperable or unsafe any guards, shields or other safety features of this product; fail, refuse or neglect to install any retrofit kits from time to time provided by Top Gun Mobility to enhance or preserve the safe use of this product.

SPECIFICATIONS

	Athena	Augustus	Caesar		Maximus		Valentina	
			Petite	Standard	Standard	Heavy Duty	Petite	Standard
Weight	45kg	57kg	46kg	57kg	57kg	68kg	46kg	57kg
Safe Working Load	150kg	150kg	150kg	150kg	150kg	250kg	150kg	150kg
Motor	Single OKIN Motor	Dual OKIN Motors	Dual Okin Motors	Dual OKIN Motors	Quad OKIN Motors	Quad OKIN Motors	Quad OKIN Motors	Quad OKIN Motors
Battery Backup	6LR61 - 9V	6LR61 - 9V	6LR61 - 9V	6LR61 - 9V	6LR61 - 9V	6LR61 - 9V	6LR61 - 9V	6LR61 - 9V
Operation Method	Handheld Remote	Handheld Remote	Handheld Remote	Handheld Remote	Handheld Remote	Handheld Remote	Handheld Remote	Handheld Remote
Seat Width	48cm	51cm	50cm	55cm	55cm	70cm	50cm	52cm
Seat Depth	51cm	51cm	51cm	51cm	51cm	51cm	51cm	51cm
Seat to Floor	49cm	49cm	49cm	49cm	49kg	49cm	45.5cm	49cm
Dimensions (L x W x H)	93 x 86 x 105cm	90 x 85 x 105cm	78 x 92 x 108cm	92 x 86 x 108cm	90 x 84 x 106cm	90 x 106 x 106cm	90 x 78 x 106cm	90 x 83 x 106cm

MANDATORY! Stay within the specified weight capacity of your Power Lift Chair. Exceeding the safe working load voids Top Gun Mobility's Power Lift Chair warranty.

GENERAL GUIDLINES

Your Power Lift Chair is a state – of – the – art life enhancement device designed to increase mobility. Please be aware that the final selection and purchasing decision regarding the type of Power Lift Chair to be used is the responsibility of you, the Power Lift Chair user, if capable of making such a decision, and/ or your healthcare professional (i.e., medical doctor, allied health professional, therapist, etc.)

There are certain situations, including some medical conditions, where you will need to practice operating the Power Lift Chair in the presence of a trained attendant. A trained attendant can be defined as a family member or healthcare professional specially trained in assisting you with performing various daily living activities while safely operating a Power Lift Chair.

Below are some precautions, tips and other safety considerations that will help you become accustomed to operating the Power Lift Chair in a safe manner.

MODIFICATIONS

Top Gun Mobility chairs are designed and engineered to provide maximum comfort and utility. However, to prevent personal injury and/ or damage to your Power Lift Chair, you should not modify, add, remove or disable any feature, part or function of your Power Lift Chair. Unauthorized modifications may also void your product's warranty.

**NOTE: Use Top Gun Mobility approved parts only for all repairs and replacements.
Your Power Lift Chair is rated for a safe working load. Refer to specific products.**

MANDATORY! Stay within the specified safe working load of your Power Lift Chair. Failure to do so may result in injuries and/ or product damage, Top Gun Mobility will not be held responsible for any such issues/ failures.

PINCH/CRUSH HAZARDS

The scissors and lift mechanisms are labelled as pinch/crush point hazards on your Power Lift Chair. Keep clear of these areas and make sure the path of motion is unobstructed.

WARNING! Do not place objects or appendages in the path of moving parts

DEGREE OF PROTECTION/MODE OF OPERATION

Class II equipment/ Type B protection against electric shock

- Degree of protection against the ingress of solid/ liquids— IPX0

ELECTROMAGNETIC AND RADIO FREQUENCY INTERFERENCE (EMI/RFI)

WARNING!

Laboratory tests have shown that electromagnetic and radio frequency waves can have an adverse effect on the performance of electrically- powered devices, such as Power Lift Chair.

Electromagnetic and Radio Frequency Interference can come from sources such as mobile phones, mobile two-way radios (such as walkie-talkies), radio stations, TV stations, amateur radio (HAM) transmitters, wireless computer links, Wi-Fi, microwave signals, paging transmitters and medium- range mobile transceivers used by emergency vehicles. In some cases, these waves can cause unintended movement or damage to the control system of electrically- powered devices. The Power Lift Chair user can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment. It is recommended that at least 3 meters (9 feet) of distance be maintained between the Power Lift Chair and any handheld equipment emitting 10W or more of output power. Refer to the manufacturer' s manual for the handheld device to determine the maximum output power of that device.

Every electrically- powered device has an immunity (or resistance) to EMI. The higher the immunity level, the greater the protection against EMI. Per EMC standards, this product has passed immunity testing and is rated as a Group 1, Class B product, meaning the Power Lift Chair uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment making the Power Lift Chair suitable for use in all establishments, including domestic establishments and hospitals.

WARNING! Be aware that cell phones, two- way radios, laptops and other types of radio transmitters may cause unintended movement of your electrically- powered device due to EMI. Exercise caution when using any of these items while operating your Power Lift Chair.

WARNING! The addition of accessories or components to the Power Lift Chair can increase the susceptibility of the chair to EMI. Do not modify your Power Lift Chair.

WARNING! Your Power Lift Chair itself can disturb the performance of other electrical devices located nearby, such as alarm systems.

NOTE: If unintended motion occurs, discontinue use of the Power Lift Chair. Contact your local retailer to report the incident.

MOTOR VEHICLE TRANSPORT

If you will be transporting your Power Lift Chair in a motor vehicle, individual components (external transformer, etc.) should be secured. The Power Lift Chair itself must also be secured (a possible hazard during vehicle braking).

IMPORTANT SAFETY INSTRUCTIONS

Read all instructions before using.

WARNING: To reduce the risk of burns, fire, electric shock, or injury to persons:

- **DANGER:** To reduce the risk of electric shock, always unplug this furnishing from the electrical outlet before cleaning.
- Unplug from outlet before putting on or taking off parts.
- Close supervision is necessary when used by (or near) children and disabled people.
- Use this furnishing only for its intended use (as described in these instructions). Do not use attachments not recommended by the manufacturer.
- Never operate this furnishing if it has a damaged cord or plug, if it is not working properly, if it has been dropped/ damaged, or dropped into water.
- Keep the cord away from heated surfaces.
- Never drop or insert any object into any opening.
- Do not use it outdoors. Just for household, indoor use only.
- Do not operate where aerosol (spray) products are being used, or where oxygen is being administered.
- To disconnect, turn all controls to the off position, then remove plug from outlet.
- **WARNING:** Risk of injury - keep children away from extended leg rest.
- **WARNING:** Risk of injury to persons - do not use this furnishing to support video equipment, such as televisions or computer monitors.
- When not in use, it is recommended the hand control be stored securely in the pocket on the outside of the arm.

SERVICING OF DOUBLE INSULATED PRODUCTS

In a double- insulated product, two systems of insulation are provided instead of grounding.

No grounding means is provided on a double- insulated product, nor is a means for grounding to be added to the product. Servicing a double- insulated product requires extreme care and knowledge of the system and is to be done only by qualified service personnel. Replacement parts for a double- insulated product must be identical to the parts they replace.


ASSEMBLY AND DISASSEMBLY

CORD AND TRANSFORMER CONNECTIONS

Once the box has been removed from around the chair, you'll notice the back is resting on top of the seat. There are a couple of items to remove from the seat portion of the chair first.


Follow the steps below:

1. Remove the zip tie holding the plastic bag with the Power Transformer, hand control and its cord on the left.
2. Remove the hand control from the plastic bag and connect it to the female plug located in the left side pocket on the outside of the chair's arm.
3. Connect the Power Transformer to the plug coming from the chair.
4. Connect the Power Cord into the Power Transformer.
5. Connect the Power Cord to an electrical outlet, near where the chair will be positioned.

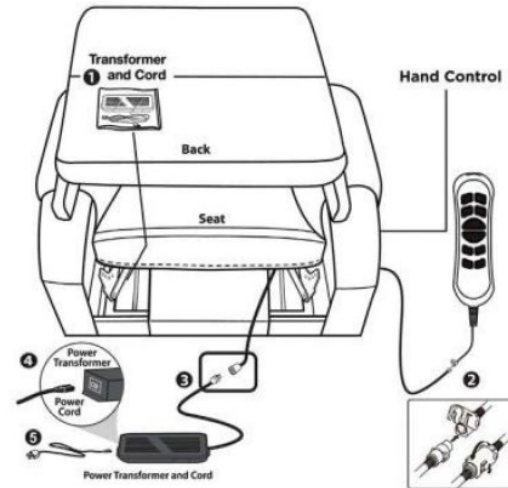

CHAIR ASSEMBLY


Attaching the chair back to the base.

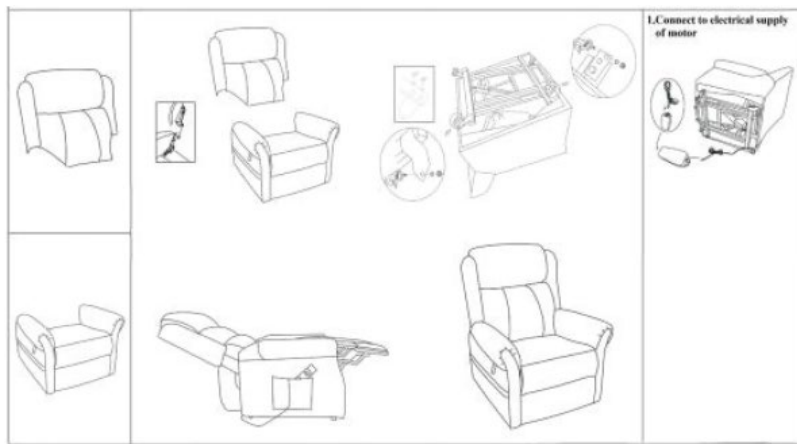

CAUTION: PLEASE READ ALL THE INSTRUCTIONS CAREFULLY BEFORE STARTING THE ASSEMBLY PROCESS. IT IS RECOMMENDED TO USE TWO PEOPLE WHEN ASSEMBLING AND REMOVING THE INSIDE BACKS TO AND FROM THE BASE OF THE FURNITURE.

1. Please do not sit on the arms or extended stool section.
2. Please do not use power stools as they can damage the chair.
3. Please do not put any part of the chair into direct sunlight.
4. Please do not place the chair near the heat.
5. Please clean/ wipe down only with a dry cloth.


OPERATING YOUR POWER LIFT CHAIR

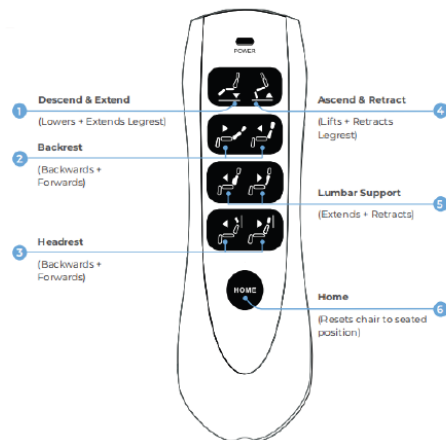

Descend and Extend Descend (Stand-to-sit): If situated in lift position, the chair will lower before extending the leg rest. Extend: Extends the leg rest to maximise comfort and relaxation.	Ascend & Retract Ascend (Lift-to-stand: Raises and tilts the chair forwards to assist users with standing. Retract: Retracts leg rest to achieve a comfortable seated position.
Backrest Backrest: Adjusts the backrest to provide postural support whatever the activity.	Lumber Support Lumber support: Adjusts the inbuilt lumbar cushion to provide tailored lower back support.
Headrest Headrest: Adjusts the headrest to help alleviate neck muscle strain, providing optimal support while reading or watching TV.	Home Home: Resets the chair to the seated position.

OPERATING YOUR 5 MOTOR POWER LIFT CHAIR

1. **Ascend & Retract Ascend:** (Lift-to-stand: Raises and tilts the chair forwards to assist users with standing. **Retract:** Retracts leg rest to achieve a comfortable seated position.
2. **Tilt-in-Space:** The chair is first tilted to its maximum position. The footrest and backrest actuators then operate together, causing the backrest to recline further. **f.** The backrest retracts first, returning the chair from the tilted position to the upright position. The footrest then retracts. Upon continued button activation, the chair transitions into the standing position.
3. **Backrest:** Adjusts the backrest to provide postural support whatever the activity.
4. **Descend and Extend Descend (Stand-to-sit):** If situated in lift position, the chair will lower before extending the leg rest. **Extend:** Extends the leg rest to maximise comfort and relaxation.
5. **Lumber Support Lumber support:** Adjusts the inbuilt lumbar cushion to provide tailored lower back support.
6. **Headrest:** Adjusts the headrest to help alleviate neck muscle strain, providing optimal support while reading or watching TV.
7. **Home:** Resets the chair to the seated position.

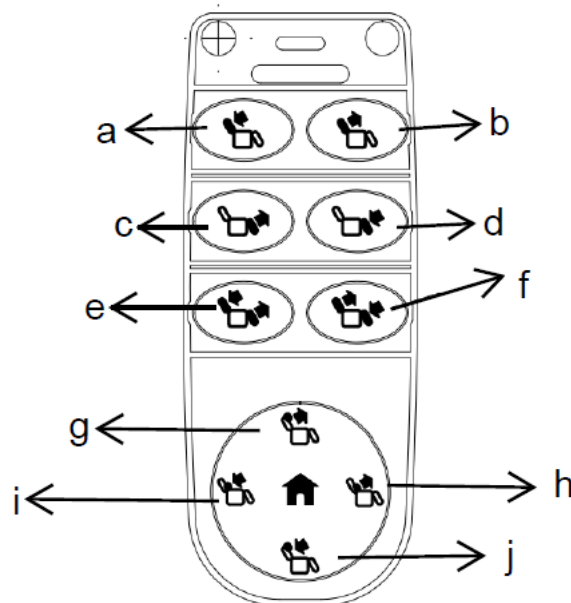

ENVIRONMENTAL PROTECTION

Recycle unwanted materials instead of disposing of them as waste. All tools and packaging should be sorted, taken to the local recycling center and disposed of in an environmentally safe way.

IMPORTANT SAFETY INSTRUCTIONS

When using an electrical appliance, basic precautions should always be followed, including the following:

This product is for use on a 12V circuit. Make sure that the product is connected to an outlet having the same configuration as the plug.

1. Always unplug the massager from the power source after use, and before cleaning.
2. Do not reach for massager if it has fallen into water. Unplug immediately.
3. An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, before putting on or taking off parts or attachments.
4. Do not use it outdoors.
5. Do not carry this appliance by supplied cord as a handle.
6. Keep the cord away from heated surfaces.
7. Close supervision is necessary when this appliance is used by on or near children, individuals or disabled persons.
8. This unit is not a toy. Children should not use it or play with it.
9. Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
10. Do not use near eyes or other highly sensitive areas.
11. To disconnect, push the "POWER" key to "OFF", make sure all functions stopped, then remove plug from outlet.
12. Keep cord away from heated surfaces.
13. Use this appliance only for its intended use as described in this manual. Do not use attachments not provided with the unit.

WARRANTY

Consumer Limited Warranty

Recliner Chairs

Top Gun Mobility has provided information throughout the user manual regarding proper usage and care of your recliner lift chair. Failure to adhere to this information will void the warranty of the recliner lift chair, its parts and the batteries supplied by Top Gun Mobility.

The following warranty is granted only to the INITIAL Consumer who has purchased our product and commences on the Date of Purchase by the Consumer from an Authorized Top Gun Mobility' Provider. The Consumer must have proof of purchase from Top Gun Mobility or the authorised recliner lift chair provider to receive service.

Note: Your selling retailer is responsible for the repairs, and should you not be able to return the product to the selling retailer or another Top Gun Mobility retailer, you may be liable for any labour charges, parts will still be supplied free of charge. You will be liable for all labour and parts costs should the repairs not be covered under warranty.

The warranty covers:
Recliner Lift Chairs

5 Year Limited Warranty:

For the life of the recliner lift chair, from the date of purchase from an authorized Top Gun Mobility' retailer, in the event of defective materials or workmanship, Top Gun Mobility will repair or replace at our option any of the following structural frame components found to be defective by an authorized Top Gun Mobility representative:

- Steel Frame
- Mechanism

2 Year Limited Warranty

For the period of three years from the date of purchase, in the event of defective materials or workmanship, Top Gun Mobility will repair or replace at our option with any new or reconditioned components any of the following drive train components found to be defective by an authorized Top Gun Mobility representative:

- Electronics
- Handset and Cable
- Fabric

NOT COVERED UNDER WARRANTY

The guarantee does not extend to those items which may need replacement due to normal wear and tear, primarily fabric or damage to the product caused by misuse or accident for which Top Gun Mobility or its agent cannot be held responsible. This warranty does not include service calls. **Transport/Call Out Costs** – As the owner you must bear the cost of transporting the unit to the retailer or should you decide to have the retailer come to you, you must bear the cost of the retailers call out fee. Your retailer may choose to waive this cost, but this will be at the retailer's discretion only and separate to Top Gun Mobility's warranty terms and conditions

COMMERCIAL USE

Any Top Gun Mobility products used in a commercial application will only be covered by a 12-month warranty from the purchase date and will be excluded from any coverage mentioned in the 3 or 5 year warranty. All other conditions in this warranty will apply.

All electronic parts, including handset and cable, have a one (1) year warranty. Servicing to the motor must be carried out by your authorised Top Gun Mobility Provider. Any attempt to fix or dismantle these items renders the guarantee void on that item.



Copyright Top Gun Mobility
User Manual & Warranty Booklet 2023
