

AIRMAX Z02

Rollator

Zoom
Mobility
EASY LIVING



USER MANUAL

FOR YOUR RECORDS

Please fill in your Zoom Mobility Rollator's information below. This information will be useful if you ever need to contact Zoom Mobility concerning your Rollator.

Name of Purchaser: _____

Address: _____

Model: *AirMax Z02 Rollator* Serial Number _____

Date of Purchase _____ Invoice Number _____

Provider Name _____

Company _____

Address _____

City _____ State _____ Postcode _____

Email: sales@topgunmobility.com.au

Thank you for purchasing your Zoom Mobility Rollator. We are honoured you have chosen Zoom Mobility for your mobility needs. Please read this user's manual carefully before using and save for future reference.

Intended Use:

The intended use of this device is to provide mobility to persons limited to a seated position that can operate a Rollator.

Standard Packing List:

1. 1 x *AirMax Z02 Rollator*
2. 1 x User Manual

If structural damage occurs during shipping, contact your supplier, and DO NOT use.

**DISCLAIMER**

We ask that you read this manual completely before operating your new Zoom Mobility Rollator. Zoom Mobility is not and cannot be held responsible for any damage or injury incurred due to improper or unsafe use of the Zoom Mobility Rollator.

Zoom Mobility specifically refuses responsibility for any bodily injury or property damage that may occur during any use that does not comply with applicable federal, state, or local laws or ordinances.

Contents

Introduction	1
SPECIFICATIONS TABLE	3
INTENDED USE	4
SAFETY GUIDELINES	4
PRECAUTIONS WHEN DRIVING	4
PART DESCRIPTION	6
ASSEMBLY INSTRUCTIONS	7
UNFOLDING THE ROLLATOR	8
FOLDING THE ROLLATOR.....	11
BRAKE FUNCTION.....	13
SAFETY INSTRUCTIONS.....	13
CLEANING	14
WARRANTY	14

Introduction

Congratulations on the purchase of your new Zoom Mobility Rollator. We are excited to provide you with one of our innovative Mobility Rollators. The Zoom Mobility Rollator *AirMax Z02 Rollator (Powerchair)* combines innovative technology with attractive designs that are also highly functional in today's world. We at Zoom Mobility know that you have chosen a Rollator that will give you years of dependable operation and will enhance the quality of your life by providing you with the mobility to experience an active daily lifestyle.

Even though your new Zoom Mobility Rollator is both user-friendly and designed for maximum manoeuvrability in even the tightest spaces, we ask that you please read, understand, and follow all the instructions and suggestions in this manual before you operate your Rollator for the first time. The safe use of your new Rollator is especially important to us.

If you feel that you do not understand the instructions and suggestions presented in this user manual, or if, for any reason, you do not feel capable of performing the activities necessary to assemble, disassemble, operate, or maintain your Zoom Mobility Rollator, please contact your local Zoom Mobility provider.

Zoom Mobility cannot be held responsible for personal injury or property damage resulting from the unsafe or the improper use of any of our personal mobility products. Also, Zoom Mobility cannot be held responsible for personal injury or property damage resulting from attempts to follow instructions, suggestions, and guidelines presented in this user manual.

Our Research and Development Department, our Quality Control Department, and our Engineering Department have used the latest product specifications and the latest product design information to manufacture your Zoom Mobility Rollator. Zoom Mobility reserves the right to implement changes into our product lines when those changes become desirable or necessary. If changes are implemented into our product line, there may be minor differences between the product you purchased and the illustrations and instructions in this user manual.

We at Zoom Mobility would appreciate hearing about the dependability of your Zoom Mobility and about the convenience of mobility it has provided for you. We would also appreciate hearing about the service you received from your local Zoom Mobility provider or representative. You can email sales@zoommobility.com.au

There are several situations, including some medical conditions, where the Rollator user will need to practice operating the Rollator in the presence of a trained attendant. A trained attendant can be defined as a family member or care professional specially trained in assisting a Rollator user in various daily living activities.

As you begin using your Rollator during daily activities you will encounter situations in which you will need some practice. Simply take your time and you will soon be in full and confident control as you manoeuvre through doorways, on and off elevators, up and down ramps, and over moderate terrain.

This manual covers the use of various models, you will note that many of the operations and adjustments are similar and as such images may depict a different model to that which you have purchased.

For more information about our products and services or to send us your comments, email sales@zoommobility.com.au

Waste Disposal



Symbol on the left indicates your product should be treated separately from domestic garbage according to local laws and regulations. When the product reaches its lifespan, please take it to the designative waste collection point run by the local authority. Sorting and recycling of this rollator will help to save the natural resources and ensure it is recycled in a way of protecting the environment.

Plastic parts, frames and tubes can be handed over to the professional recycling units for disposal after reaching its lifespan.

SPECIFICATIONS TABLE

AirMax Z02 Rollator.

MODEL DETAILS	<i>AirMax Z02 ROLLATOR</i>
Weight	6kg
Safe Working Load (SWL)	136kg
Safe Working Load (SWL) of Bag	5kg
Seat (W x H)	43.4 x 55.2cm
Handle Height	88-98.2cm
Overall (W x D)	63 x 69cm



** Specifications are correct at time of print and are subject to change without notification

INTENDED USE

The AirMax Z02 Rollator is an indoor/outdoor rollator intended to be used for people who are experiencing generalized weakness, difficulty with balance, a lower extremity injury or surgery that need additional support to maintain balance and stability while walking indoors or used for rehabilitation. It is designed for a **safe working load of 136kg**. Please do not use this rollator if physical restrictions like weak arms, central vestibular disequilibrium syndrome, severe equilibrium disorders, functional perceptual disorders, or extremity loss.

SAFETY GUIDELINES

Where applicable.

Please read this manual carefully before first using the AirMax Z02 Rollator. It contains important safety instructions and valuable tips on correctly using and caring for this rollator. Instructions and valuable tips on correctly using and caring for this rollator. If you have any questions or require additional information, please contact the nearest distributor for help.

This chapter contains the precautions you need to know when using the AirMax Z02 2n1 Rollator. Both people in the rollator and caregivers should understand these precautions.

PRECAUTIONS WHEN DRIVING

- Before using the rollator, please decide if you require a caregiver. You can refer to the advice of physiotherapists, doctors, and experts.
- Before you become familiar with the operation, please practice driving in flat and safe areas with a caregiver.
- Do not use your rollator if any of the parts are damaged, cracked or missing.
- Do not modify the chair or any spare parts, as this may void all warranties.
- Do not use the rollator when you are not feeling well, or under influence.
- Loose parts, loose bolts, tyre abrasion and other problems may cause accidents.
- Please be sure to get your rollator (powerchair) serviced by a Zoom Mobility Retailer near you once a year.
- When the passenger is sick, it is necessary to use the rollator (powerchair) with caution. Do not use the rollator when it is not suitable for your physical condition. Please put the brakes on the rollator before you make any adjustments.

- Before the rollator moves, check the ground for bumps and obstacles, it may cause a fall, component damage, etc.
- When you are in the rollator, please keep a correct posture and do not take a posture that is easy to fall or slip out. Otherwise, you may lose balance and fall.
- Be careful when you encounter poor road conditions or moving forward on a ramp. Otherwise, you may lose balance and fall. The same goes for waiting near escalator entrances, or elevator doors. When you come across railroad tracks, keep straight ahead of them.
- When you cross a railway track, go over straight and not on an angle. If you go across on an angle, your wheel may get stuck in the gap of the track.
- Please do not put anything on the armrest or frame of the rollator. (If you hold the armrest, handles, seat cushions or others, it may cause damage to the components or other accidents.)
- Please try not to brake on a slope. If you must stop, employ brakes on each side at the same time.
- Do not use the product as a ladder.
- Please do not allow children to operate, or two or more people use it at the same time.
- When the frame is bent, cracked, or the seat cushion, seat belt and other parts are broken, do not use. (Stop using the rollator and contact your nearest Zoom Mobility retailer.)
- Please use this product with shoes on, otherwise accidents may occur. Please do not use the product if the weight exceeds the safe working load.
- Keep enough distance when driving next to a fence or tree so that branches and others cannot encounter the rear tyres.
- Do not drive with heavy object hanging on the handle.
- Do not use the rollator on slopes with inclines greater than 10% (6°) and lean forward to improve your centre of gravity when going uphill.
- Do not use in icy areas.
- If there are concerns with your rollator, please contact your nearest Zoom Mobility Retailer.

PART DESCRIPTION

1) Handle	2) Backrest
3) Seat	4) Shopping Bag
5) Main Frame	6) Front Wheel
7) Rear Wheel	



ASSEMBLY INSTRUCTIONS

1. Remove the item from the box and check the contents and the packaging for damage or completeness.

1 x Main Frame	1 x Shopping Bag
1 x Backrest	2 x Front Wheel
1 x QR Code Card	2 x Handrail with rear leg tube



UNFOLDING THE ROLLATOR

Insert front wheel into the main frame, you can hear a "click" sound and grey cover is pushed out.



Rear leg tube with wheel outwards, insert it into the main frame until the snap button pops out the hole on main frame.



Attach the handrail handle into the main frame. And push the button on the handle holder part to choose the suitable handle height. Then repeat for the other side. Check that both handles and make sure they are at the same height and

stable.



Press down the tubes on both of seat sides until the frame unfolds and you can hear a "click". Now the locking device is in position.



Attach the shopping bag on seat extended frame.



Attach the backrest onto handle tube, insert the bolt into backrest bracket hole from inner side and tighten with a knob outside. Repeat this for the other side. Make sure the knobs are outside. Make sure the backrest height is suitable for use.



Now the rollator is ready for use.

FOLDING THE ROLLATOR

Pull up the strap at the seat until the frame folds together from side to side.



- To detach the front wheel, press the top grey button on the front wheel, then take the wheel off.



- To detach the rear leg tube, press the snap button then remove the rear leg tube off.



- To detach the handrail tube, press the grey button then remove the handrail tube off.



BRAKE FUNCTION

Running brake: Squeeze the brake handle upwards to break the rollator.

Parking brake: Squeeze the brake handle downwards until hearing "click" to lock the rollator.

SAFETY INSTRUCTIONS

Assess the brakes before using and check the brakes after each height adjustment.

2. Do not hang any objects on the handle or brake handle.
3. Before each use, check whether the rollator is completely unfolded and whether the handles on both sides are at the same height.
4. Do not use the rollator on stairs, but only on level ground! All four wheels should stand on the floor, otherwise there is danger of tipping!
5. ***Never release the brakes when seated on the rollator and always activate the brakes when not using the rollator!***
6. The rollator capacity is 136 kg! The carrier bag can be loaded up to a max. 5 kg and is intended solely for the transport of goods!
7. Do not use a defective rollator and contact the authorized dealer if you notice any material changes or signs of strong wear and tear.

CLEANING

You should clean/disinfect the rollator at regular intervals (mandatory at the time of user change).

- * Use a mild soap without abrasive additives and a soft cloth to clean the rollator.
- * Do not use abrasive or corrosive cleaning agents or high-pressure and steam cleaners!
 - * Make sure that no dirt is deposited on the tires or brakes. This can be removed with a soft brush and warm water.
- * Dry the rollator after cleaning. Also dry your rollator after you have been, for example, in a rainstorm.

WARRANTY

Consumer Limited Warranty

Rollators

Zoom Mobility has provided information throughout the user manual regarding proper usage and care of your Rollator. Failure to adhere to this information will void the warranty of the Rollator, its parts and the batteries supplied by Zoom Mobility.

The following warranty is granted only to the INITIAL Consumer who has purchased our product and commences on the Date of Purchase by the Consumer from an Authorized Zoom Mobility' Provider. The Consumer must have proof of purchase from Zoom Mobility or the authorised Rollator provider to receive service.

Note: Your selling retailer is responsible for the repairs, and should you not be able to return the product to the selling retailer or another Top Gun Mobility retailer, you may be liable for any labour charges, parts will still be supplied free of charge. You will be liable for all labour and parts costs should the repairs not be covered under the warranty.

The warranty covers the following Rollator models unless otherwise noted:

AirMax Z02 Rollator (Powerchair) Rollator

1 Year Limited Warranty (Repairing):

- ◆ Frame

1 Year Limited Warranty (Replacement):

For the life of the Rollator, from the date of purchase from an authorized Zoom Mobility' retailer, in the event of defective materials or workmanship, Zoom Mobility will repair or replace at our option any of the following structural frame components found to be defective by an authorized a Zoom Mobility representative:

- ◆ Handle Sleeve & Brake Lever

The warranty of this product is based on the normal use and maintenance described in this manual. It does not cover device failure due to owner's misuse or negligence maintenance.

Warranty Exclusions:

Zoom Mobility does not provide warranty on any of the following items which may require replacement due to the normal wear and tear of day-to-day usage:

- ◆ Wheels
- ◆ Storage Bag
- ◆ Seat
- ◆ Plastic Parts (vulnerable parts)

If something is damaged during use, please contact the Retailer.

This warranty also excludes for the following:

- ◆ Loss or theft of components

- ◆ Damage caused by:
 - Abuse, misuse, accident, or negligence.
 - Improper operation, maintenance, or storage.
 - Commercial use or use other than normal.
 - Exceeding the specified weight capacity of the unit.
 - The product is beyond the warranty period.
 - Repairs and/or modifications made to any part without the specific consent of Zoom Mobility.
 - Accessories other than those supplied or approved by Zoom Mobility.
 - Failure to adhere to the product instructions.
 - Acts of Nature (Force majeure), such as lightning strikes, cyclones, earthquakes, etc.
 - An accident occurs causing damages to a part/s.
 - Circumstances beyond the control of Zoom Mobility.
- ◆ ANY PARTS ALTERED OR REPAIRED BY AN UNAUTHORIZED PERSON.

There is no other express warranty.

Implied warranties including those of merchantability and fitness for a particular purpose are excluded. Liabilities for consequential damages are excluded. This warranty gives you specific rights and you may also have other rights which may vary from state to state.



Technical Service, please scan the following QR code.



Copyright Top Gun Mobility
User Manual & Warranty Booklet 2023