

# TOPFLYTE

## Wheelchair

**TopGun**  
**Mobility**  
WHEELS FOR LIFE



# USER MANUAL





## Table of Contents

|                                                           |   |
|-----------------------------------------------------------|---|
| INTRODUCTION .....                                        | 1 |
| INTENDED USE .....                                        | 1 |
| FEATURES.....                                             | 1 |
| MODEL FEATURES.....                                       | 2 |
| OPERATIONAL INSTRUCTIONS – Unfolding the Wheelchair ..... | 4 |
| OPERATIONAL INSTRUCTIONS – Folding the Wheelchair .....   | 5 |
| SAFETY .....                                              | 6 |
| General .....                                             | 7 |
| Care & Maintenance .....                                  | 7 |
| Modifications .....                                       | 7 |
| WARRANTY .....                                            | 8 |

---





## INTRODUCTION

**CONGRATULATIONS** on the purchase of your new Top Gun TOPFLYTE Wheelchair. We are excited to provide you one of our innovative, lightweight, and compact wheelchairs.

This Owner's Manual contains important information concerning the proper maintenance and safe operation of your wheelchair. Please read this manual thoroughly and carefully prior to operating your new wheelchair to familiarise yourself with all operational instructions.

## INTENDED USE

The TOPFLYTE Wheelchair is intended for indoor or outdoor use with an attendant. This lightweight foldable wheelchair is designed for carrying a **maximum weight capacity up to 115kg** with comfort with easy-to-use features. When travelling on the horizontal ground, the attendant can easily push the wheelchair by holding the handgrips.

## FEATURES

- Ultra-light, weighing only 8.9kg.
- Soft seat and backrest for comfort.
- Powder coated aluminium frame to prevent corrosion and maximise lifespan.
- Swing away footrest design to reduce accidents during user transfer.
- Attendant hand brakes ensure safety.
- Seat Belt included.
- Half folding backrest design makes for easy transportation and compact storage.

## MODEL FEATURES

1. Handles
2. Footrests
3. Leg Rest
4. Footrest Height Adjusting Pin
5. Release Clip for Leg Rest
6. Backrest Unlock Clips
7. Main Frame
8. Lockable Park Brakes
9. Attendant Handbrakes
10. Front Castor
11. Back Wheel
12. Seat
13. Backrest
14. Leg Support



### NOTE:

Check all parts for shipping damage. In case of shipping damage, Do NOT use.

Contact authorised Top Gun Mobility dealer for further instruction.

## SPECIFICATION TABLE

|                             | TOPFLYTE  |
|-----------------------------|-----------|
| Seat Width                  | 46cm      |
| Seat Depth                  | 40cm      |
| Seat Height (from floor)    | 47.5cm    |
| Backrest (from seat)        | 40.6cm    |
| Seat Backrest (from floor)  | 85cm      |
| Armrest Height (from floor) | 70.5cm    |
| Front Wheels                | 15cm (6") |
| Rear Wheels                 | 20cm (8") |
| Net Weight                  | 8.9kg     |
| Safe Working Load (SWL)     | 115kg     |

# OPERATIONAL INSTRUCTIONS – Unfolding the Wheelchair

## 1. Unfolding the Backrest

Lift up both the handles towards the frame until you hear a click locking it in place.

## 2. Unfold the Wheelchair

Firmly holding both handles, push outwards until the frame is half open. Place both hands on the side of the seat and push downwards to fully expand the wheelchair.

## 3. Attaching the Leg Rests

Position the leg rest parallel to the side of the frame, with the footrest facing the back wheels and the silver clip facing away from the frame. Place the leg rests securely on the 2 silver pins located above the front castors. Then swivel the leg rest forward until you hear the silver clip locking it against the frame. Push both footplates down until they are parallel to the ground.

## 4. Adjusting the Height of Footrests

The footrest has 5 height level options. Press the 2 silver pins to unlock the footrest and then adjust it to the desired level.



# OPERATIONAL INSTRUCTIONS – Folding the Wheelchair

## 1. If Want to Remove Leg Rests

Undo the silver clip by pulling the clip towards the opposite leg rest and remove each leg rest by lifting it up from the pins.



## 2. Folding the Wheelchair

Pull the footplates up in readiness to fold the wheelchair. Grab the front and back of the seat and lift upwards to collapse the wheelchair halfway. Then gently push both the arms inwards to fully fold and collapse the wheelchair.

## 3. Folding the Backrest

Pull the 2 black clips located behind each armrest to unlock the frame. Push down both the handles towards the back wheels to fold.



## SAFETY

**DO NOT** assemble or use this wheelchair without first reading and understanding the instructions.

**Stability** - The TOPFLYTE Wheelchair is designed to ensure safety and stability, provided that you maintain the center of gravity while using it. To preserve the chair's stability, it's important to avoid sudden movements that could lead to a shift in your center of gravity. To ensure the safe operation of the wheelchair, these instructions must be followed.

**Hand Brakes** - For safety, this wheelchair comes with attendant handbrakes to provide better control.

**Inclined Surface** - When attempting to navigate inclined surfaces, including ramps, with this wheelchair, it's important to consider your strength, capabilities, and limitations. For correct techniques, a qualified healthcare professional may be able to provide product demonstrations and valuable advice.

**NEVER** put your hands, fingers, feet or legs into any moving part. **When making adjustments** (i.e. adjusting the leg rest height etc.), always take care that your hands or fingers are not trapped between any of the moving parts



**LOAD RESTRICTIONS** – Maximum carrying capacity is 115kg.



**TYRE INFLATION** – Only if your wheelchair is equipped with pneumatic tyres. If your TOPFLYTE Wheelchair is equipped with pneumatic tyres, check the tyre pressure at least once a week. Proper inflation pressures will prolong the life of your tyres and help ensure smooth operation.

## General

**MANDATORY!** Do not operate your new TOPFLYTE Wheelchair for the first time without reading and understanding this user manual completely. Healthcare Professionals and/or the authorized dealer to give instructions for the use of the product.

## Care & Maintenance

The wheelchair should be regularly wiped using warm water and mild detergent. Always ensure that the wheelchair is dried after use to prevent corrosion and maintain its appearance.

1. Ensure that ALL attaching hardware is always secure.
2. Check if the wheels are installed and secured properly.
3. Make sure all pins are fully protruded through adjustment holes of each height adjustment tube with a clicking sound. Always test to see that the wheelchair and attachments are properly and securely locked in place before using.
4. Replace any broken, damaged, or worn items immediately. Store in a dry environment.

## Modifications



We have designed and engineered your TOPFLYTE Wheelchair to provide maximum mobility and utility. Under no circumstances should you modify, add, remove, or disable any feature, part, or function of your wheelchair.

**WARNING!** Do not modify your TOPFLYTE Wheelchair in any way not authorized by the manufacturer. Do not use accessories if they have not been tested or approved by the manufacturer.

## WARRANTY

This is to certify that your TOP GUN MOBILITY product offers a 5 Year limited Warranty from the date of original purchase. Should a defect or fault arise within this period the Dealer, from where the product was originally purchased, should be notified immediately. Failure to register the fault may invalidate the warranty.



The Dealer from where the product was purchased is responsible for any warranty repairs. Top Gun Mobility warrants that if a fault occurs within this period due to faulty workmanship or materials that the product will be repaired (or replaced at the Company's discretion) free of charge. ***Any unauthorised modifications will invalidate the warranty.***

| <b>FIVE-YEAR LIMITED WARRANTY</b> | <b>12 MONTH LIMITED WARRANTY</b>                                                                                                                                                                                                             |
|-----------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Structural frame ONLY             | Your Top Gun TOPFLYTE Wheelchair is fully guaranteed for 12-months from the date of purchase against faults arising due to defects in manufacture or materials. This warranty does not detract from but is in addition to your legal rights. |

### COMMERCIAL USE

Any Top Gun Mobility products used in a commercial application will only be covered by a 12-month warranty from the purchase date and will be excluded from any coverage mentioned in the warranty relating to 5 (five) warranty. All other conditions in this warranty will apply.

### NOT COVERED UNDER WARRANTY

The guarantee does not extend to those items which may need replacement due to normal wear and tear or damage to the product caused by misuse or accident for which Top Gun Mobility or its agent cannot be held responsible. This warranty does not include service calls.





Copyright Top Gun Mobility  
Owner's Manual & Warranty Booklet 2023