

AIRMAX Z06 Rollator

**Zoom
Mobility**
EASY LIVING



USER MANUAL

FOR YOUR RECORDS

Please fill in your Zoom Mobility Rollator's information below. This information will be useful if you ever need to contact Zoom Mobility concerning your Rollator.

Name of Purchaser: _____

Address: _____

Model: *AirMax Z06* Rollator Serial Number _____

Date of Purchase _____ Invoice Number _____

Provider Name _____

Company _____

Address _____

City _____ State _____ Postcode _____

Email: sales@zoommobility.com.au

Thank you for purchasing your Zoom Mobility Rollator. We are honoured you have chosen Zoom Mobility for your mobility needs. Please read this user's manual carefully before using and save for future reference.

Intended Use:

The intended use of this device is to provide mobility to persons limited to a seated position that are capable of operating a Rollator.

Standard Packing List:

1. 1 x *AirMax Z06* Rollator
2. 1 x User Manual

If structural damage occurs during shipping, contact your supplier, and DO NOT use.

**DISCLAIMER**

We ask that you read this manual completely before operating your new Zoom Mobility Rollator. Zoom Mobility is not and cannot be held responsible for any damage or injury incurred due to improper or unsafe use of the Zoom Mobility Rollator.

Zoom Mobility specifically disclaims responsibility for any bodily injury or property damage that may occur during any use that does not comply with applicable federal, state, or local laws or ordinances.

Contents

Introduction	1
Specifications Table	3
Intended Use.....	4
Safety Guidelines	4
Instructions Before Use.....	6
Opening	7
Folding	8
Adjusting the handlebar.....	8
Using the Brake/s.....	9
Maintenance	9
a. General Information.....	9
b. Check List	10
c. Cleaning and Disinfection.....	10
d. Storage and Transportation Condition.....	10
e. Ambient Temperature - Transportation & Storage	11
f. Disposal.....	11
Warranty	12

Introduction

Congratulations on the purchase of your new Zoom Mobility Rollator. We are excited to provide you with one of our innovative Mobility Rollators. The Zoom Mobility Rollator *AirMax Z06* combines cutting edge technology with attractive designs that are also highly functional in today's world. We at Zoom Mobility know that you have chosen a Rollator that will give you years of dependable operation and will enhance the quality of your life by providing you with the mobility to experience an active daily lifestyle.

Even though your new Zoom Mobility Rollator is both user-friendly and designed for maximum manoeuvrability in even the tightest spaces, we ask that you please read, understand, and follow all the instructions and suggestions in this manual before you operate your Rollator for the first time. The safe use of your new Rollator is very important to us.

If you feel that you do not understand the instructions and suggestions presented in this user manual, or if, for any reason, you do not feel capable of performing the activities necessary to assemble, disassemble, operate, or maintain your Zoom Mobility Rollator, please contact your local Zoom Mobility provider.

Zoom Mobility cannot be held responsible for personal injury or property damage resulting from the unsafe or the improper use of any of our personal mobility products. Also, Zoom Mobility cannot be held responsible for personal injury or property damage resulting from attempts to follow instructions, suggestions, and guidelines presented in this user manual.

Our Research and Development Department, our Quality Control Department, and our Engineering Department have used the latest product specifications and the latest product design information to manufacture your Zoom Mobility Rollator. Zoom Mobility reserves the right to implement changes into our product lines when those changes become desirable or necessary. If changes are implemented into our product line, there may be minor differences between the product you purchased and the illustrations and instructions in this user manual.

We at Zoom Mobility would appreciate hearing about the dependability of your Zoom Mobility and about the convenience of mobility it has provided for you. We would also appreciate hearing about the service you received from your local Zoom Mobility provider or representative. You can email sales@zoommobility.com.au

There are several situations, including some medical conditions, where the Rollator user will need to practice operating the Rollator in the presence of a trained attendant. A trained attendant can be defined as a family member or care professional specially trained in assisting a Rollator user in various daily living activities.

As you begin using your Rollator during daily activities you will probably encounter situations in which you will need some practice. Simply take your time and you will soon be in full and confident control as you manoeuvre through doorways, on and off elevators, up and down ramps, and over moderate terrain.

This manual covers the use of various models, you will note that many of the operations and adjustments are similar and as such images may depict a different model to that which you have purchased.

For more information about our products and services or to send us your comments, email sales@zoommobility.com.au

Waste Disposal



Symbol on the left indicates your product should be treated separately from domestic garbage according to local laws and regulations. When the product reaches its lifespan, please take it to the designative waste collection point run by the local authority. Sorting and recycling of this rollator will help to save the natural resources and ensure it is recycled in a way of protecting the environment.

Plastic parts, frames and tubes can be handed over to the professional recycling units for disposal after reaching its lifespan.

Specifications Table

AirMax Z06 Indoor Rollator compliant with the requirements specified in ISO 13485.

The four-wheeled walking aid can be folded up for storage and transportation. Front wheels are steering wheels. Rear wheels are provided with a brake system for slowing down and for locking the wheels.

MODEL DETAILS	AIRMAX Z06 ROLLATOR
APPROVED FOR INDOOR USE ONLY	
Brake Method	Pull Up Handlebar Brake
Construction Material	Aluminium
Dimensions	
Folded Dimensions (WxDxH) – cm	56 x 26 x 93cm
Depth (overall)	66cm
Width (overall)	56cm
Height – Minimum to Maximum (3 x adjustments)	83.5cm to 94.5cm
Seat/Carry Bag	
Tray Depth	29cm
Tray Width	39cm
Tray Height	68cm
Weight Capacity - Rollator	100kg
Weight Capacity – Carry Bag	5kg
Weight – Net	6kg
Wheel Size (Front & Rear)	7" PVC



** Specifications are correct at time of print and are subject to change without notification

Intended Use

The AirMax Z06 is an indoor rollator intended to be used for people who are experiencing generalized weakness, difficulty with balance, a lower extremity injury or surgery that need additional support to maintain balance and stability while walking indoors or used for rehabilitation. It is designed for a **maximum body weight of 100kg**. The rollator only provides supports to your legs when walking. Please do not use this rollator if physical restrictions like weak arms, central vestibular disequilibrium syndrome, severe equilibrium disorders, functional perceptual disorders, or extremity loss.

Safety Guidelines

Where applicable.

Please read this manual carefully before first using the AirMax Z06. It contains important safety instructions and valuable tips on correctly using and caring for this rollator. Instructions and valuable tips on correctly using and caring for this rollator. If you have any questions or require additional information, please contact the nearest distributor for help.

1. This product is designed and produced solely for single person use with a maximum weight of 100kg. It is not intended for transport of goods or to be used as wheelchairs, carts etc. The rollator will compensate the lack of adequate support by the legs by leaning on, or moving with, with usage of the upper limbs or chest. The rollator will provide better stability, support, and safety during walking.
2. Do not hang anything on the handle or the other parts to this product.
3. DO NOT use on stairs or on escalators or stand on this product.
4. DO NOT let children play with the product.
5. Make sure the rollator is correctly unfolded, the seat is down, and the frame is locked securely. Check the wheels are not impeded by any foreign objects.
6. Avoid walking against obstacles, both forwards and backwards.
7. It is recommended to consult your doctor, physiotherapist, or occupational therapist to determine whether this product is suitable for you.
8. Maximum load capacity for the storage bag is 5kg. There may be some risks when the maximum weight is exceeded.

9. The rollator can be used indoors. Ensure all four wheels are touching the ground. Do not use the rollator on rocky, bumpy, or sandy terrain, or places with a steep inclination. These terrains will affect the body balance and safety.
10. A caregiver/attendant should assist in the height adjustment of the rollator for maximum support and correct brake activation, the fixture of the screw.
11. DO NOT install or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the Warnings, Cautions or Instructions, contact your healthcare professional, dealer, or technical personnel before attempting to install this equipment – otherwise, injury or damage may occur.
12. Each individual should always consult with their physician or therapist to determine proper adjustment and usage.
13. DO NOT go up or down inclines greater than 15° or diagonally across inclines greater than 5° when using as a traditional rollator with armrests locked in horizontal position.
14. DO NOT ride the Walker with all your weight on the armrests or lean disproportionately on one of the armrests.
15. Rollators are **NOT** intended to be self-propelled.
16. Always ensure brake cables are flush along leg extensions. Brake cables should always be secured with the supplied tie wrap to the side braces. After any height adjustment, reposition brake cables so that any excess cable loops out the front of the rollator. Always ensure screws are fastened to the connector, so that the connector can't move when forced. To assure the correct position of the quick release/assembly kits, they will not fall even with the opposite force. Ensure all snap buttons fully protrude through adjustment holes. Failure to follow these warnings can result in personal injury.
17. Care should be taken to ensure that **ALL** handles, height adjustment locks, wheels, brakes or any other functional or moving parts are in good working order before operation.
18. All wheels **MUST** be in contact with the floor at **ALL TIMES** during use. This will ensure the rollator is properly balanced. The brakes **MUST** be in the locked position **BEFORE** using the seat.
19. When using the rollator in a stationary position, the hand brakes **MUST** be locked.
20. Always pay attention to the surface/road conditions and ensure there is no obstacle in front of you when using the rollator.

21. Please hold the handles with both hands when using.
22. Please be aware of that some parts of rollator may become very hot or cold due to the factors like solar radiation. Please be careful when touching (if applicable).
23. Check whether the handle adjustment mechanism is properly locked, and the handle is reliable before using the rollator.
24. Using the rollator with wet hands may cause loss of control or balance.
25. To use the rollator on a motorway is prohibited.
26. Please note that the proper care and maintenance of the rollator can prolong its service life. But defective or neglected care and lack of maintenance of the rollator will limit liability.

Instructions Before Use

ATTENTION! Please read this manual carefully before use and follow the instructions to use the rollator in the correct way and avoid physical injury or damage. Ensure all components are in good condition to ensure your safety.

- Carefully remove all components from the carton and check the product for any defect; do not use a defective product.
- Ensure a healthcare professional has assessed that the product gives the required support and that its adjustments are appropriate for the user.
- Ensure the carer or user has been appropriately trained by a healthcare professional on how to use the product correctly and safely.
- Do not walk on the damp floor or where there is insufficient light and/or obstructions to avoid slipping or tripping.
- Do not wear socks without shoes, slippers or high heels when walking with the rollator.
- Steady yourself before walking using a slow small to medium stride while looking forward (not down).
- The users should be familiar with all features, intended use, and maintenance of this rollator.

Opening

- a. Take the rollator out of the carton and put it upright on the floor. Support the handle with one hand and hold the front edge of the tray/basket/bag with the other hand. Pull the rear leg tube to the direction as shown in Figure 1 to open the rollator frame fully. The tray is in place when you hear a clear “click” sound.
- b. Push down on the tray frame and the handle to fix in position as shown in Figure 2.

Figure 1


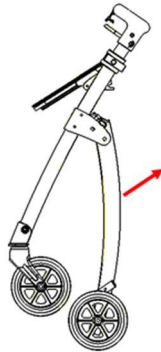




Figure 2

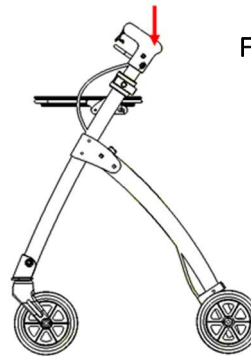

Note: The figures are without a bag for better illustration.

Folding

- Grasp the folding bar shown in Figure 3 to fold the tray frame.
- Press the folding button shown in Figure 4.
- Lift folding bar and then the rollator can be foldable.

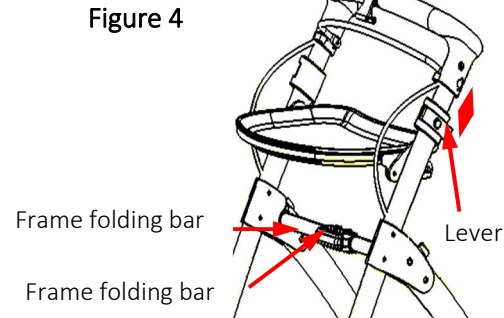
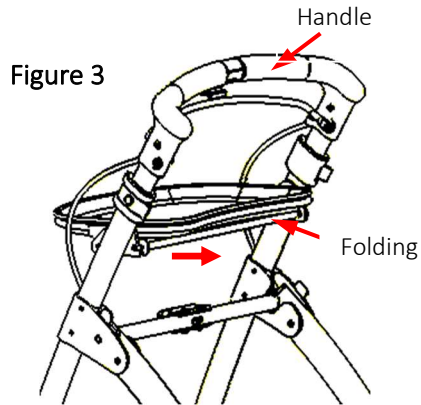


Adjusting the handlebar

- Open the lever shown in Figure 4; slide the handle to the desired height.
- Push the lever back through the tubing holes until fully clicked in position. Ensure that the handles are secure and stable before use.

Using the Brake/s

- a. Grasp both handles and brake bar, pull up the brake bar as shown in Figure 5, fasten the brake bar with red brake lever as shown in Figure 6 with a clear click.
- b. Remove the brake; rotate the red lever in the opposite direction.

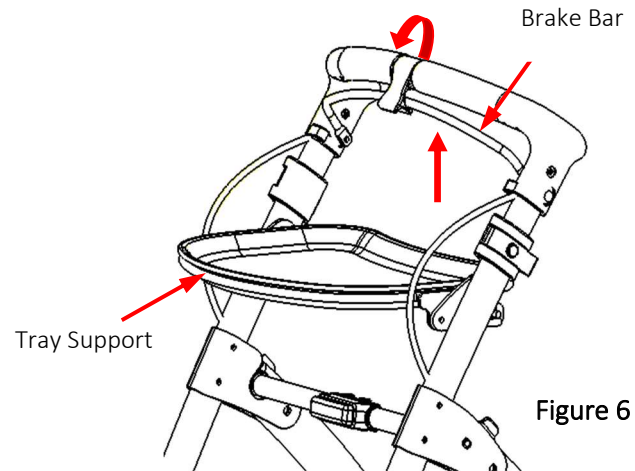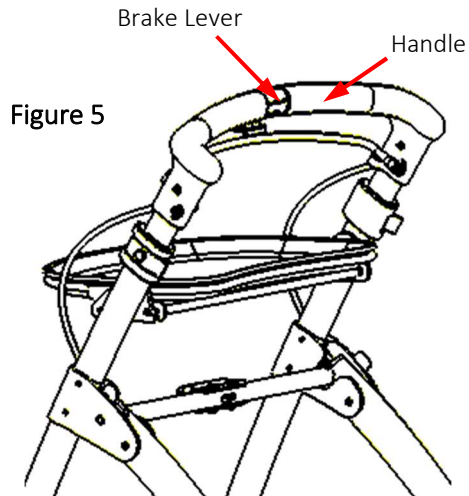

Maintenance

- a. **General Information**
 - i. Maintenance should be carried out regularly.

b. Check List

- i. Check the fit and tightness of all nuts, bolts and knobs.
- ii. Ensure all snap buttons fully protrude through adjustment holes of each tube.
- iii. Ensure that hand grips are tight.
- iv. Verify brake operation.
- v. Inspect the frame and loaded parts for cracks.
- vi. Inspect the wheels' tightness, wear, and damage.
- vii. Make sure that the wheels are free of hair, lint, and other debris.

c. Cleaning and Disinfection

- i. Take precautions for yourself and use appropriate protective equipment.
- ii. Wrong fluids or methods can harm or damage the product.
- iii. All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- iv. Never use corrosive fluids (alkaline, acid etc.) or abrasive cleaning agents.
- v. Clean and air dry.

Attention

- I. Do not use high pressure cleaner to clean this rollator.
- II. Do not use volatile chemical solvent like heavy concentrations of alcohol, ether, heavy concentrations of acids or Chlorinated hydrocarbons e.g. banana oil, gasoline, kerosene and wax with a sponge and brush which can be abrasive.
- III. All parts must be thoroughly dried before using.

d. Storage and Transportation Condition

- i. Should be stored in clean, ventilated, and dry place.
- ii. Rainproof, moisture-proof, and handle with care during transportation.

e. Ambient Temperature - Transportation & Storage

- a. Ambient temperature range: -40°C to 65°C.
- b. Relative Humidity: 0% to 90%.
- c. Do not store other objects on the rollator to prevent damage caused by long-term weight bearing down.

f. Disposal

- i. Disassemble the products and its components, separate and recycle the different materials individually.
- ii. The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in your country.

pWarranty

Consumer Limited Warranty

Rollators

Zoom Mobility has provided information throughout the user manual regarding proper usage and care of your Rollator. Failure to adhere to this information will void the warranty of the Rollator, it's parts and the batteries supplied by Zoom Mobility.

The following warranty is granted only to the INITIAL Consumer who has purchased our product and commences on the Date of Purchase by the Consumer from an Authorized Zoom Mobility' Provider. The Consumer must have proof of purchase from Zoom Mobility or the authorised Rollator provider to receive service.

The warranty covers the following Rollator models unless otherwise noted:

AirMax Z06 Rollator

1 Year Limited Warranty (Repairing):

- ◆ Frame

1 Year Limited Warranty (Replacement):

For the life of the Rollator, from the date of purchase from an authorized Zoom Mobility' dealer, in the event of defective materials or workmanship, Zoom Mobility will repair or replace at our option any of the following structural frame components found to be defective by an authorized a Zoom Mobility representative:

- ◆ Handle Sleeve & Brake Lever

The warranty of this product is based on the normal use and maintenance described in this manual. It does not cover device failure due to owner's misuse or negligence maintenance.

Warranty Exclusions:

Zoom Mobility does not provide warranty on any of the following items which may require replacement due to the normal wear and tear of day-to-day usage:

- ◆ Wheels
- ◆ Seat
- ◆ Plastic Parts (vulnerable parts)
- ◆ Storage Bag

If something is damaged during use, please contact the Dealer.

This warranty also excludes for the following:

- ◆ Loss or theft of components
- ◆ Damage caused by:
 - Abuse, misuse, accident, or negligence.
 - Improper operation, maintenance, or storage.
 - Commercial use or use other than normal.
 - Exceeding the specified weight capacity of the unit.
 - The product is beyond the warranty period.
 - Repairs and/or modifications made to any part without the specific consent of Zoom Mobility.
 - Accessories other than those supplied or approved by Zoom Mobility.
 - Failure to adhere to the product instructions.
 - Acts of Nature (Force majeure), such as lightning strikes, cyclones, earthquakes, etc.
 - An accident occurs causing damages to a part/s.
 - Circumstances beyond the control of Zoom Mobility.
- ◆ ANY PARTS ALTERED OR REPAIRED BY AN UNAUTHORIZED PERSON.

There is no other express warranty.

Implied warranties including those of merchantability and fitness for a particular purpose are excluded. Liabilities for consequential damages are excluded. This warranty gives you specific rights and you may also have other rights which may vary from state to state.



Technical Service, please scan the following QR code.



Copyright Top Gun Mobility
User Manual & Warranty Booklet 2023